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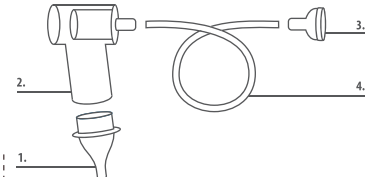
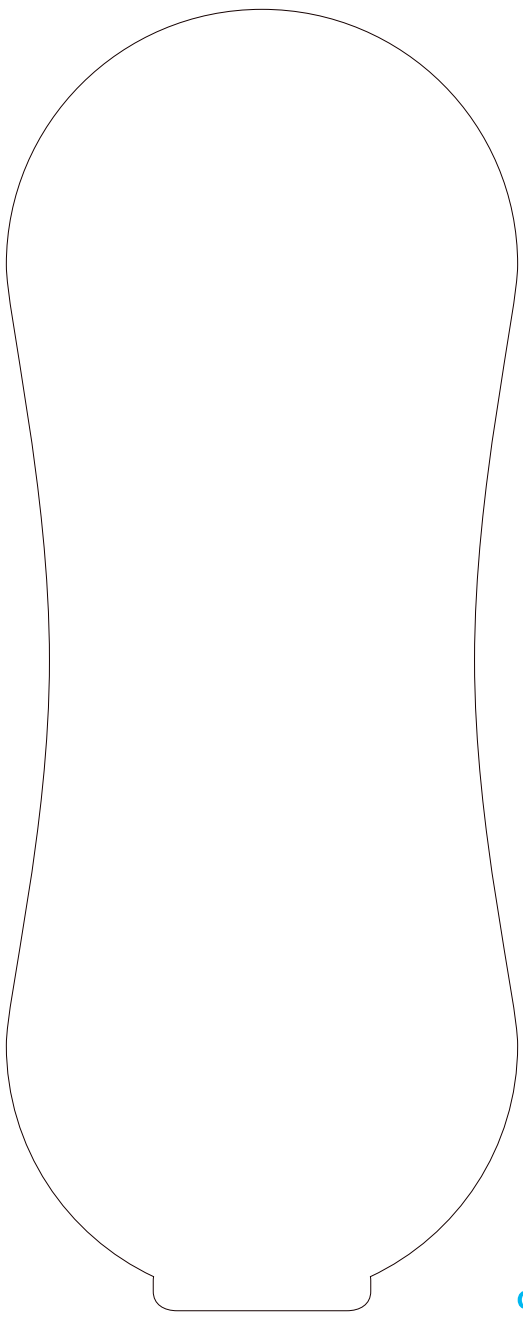
adjustable suction power
Aspirator do nosa
kontrolowana siła ssania

Cat. No 485

EN The nasal aspirator. Important for your baby's health and safety: **WARNING!** Please check the product before each use. If you notice any signs of wearing out or damage, please discard the product. Please keep the instruction manual for future reference, as it contains important information.

User's manual: Put the child in a comfortable position on his/her back, e.g. on a changing pad, bed or your lap. You should be facing the child's nose. Hold his or her head to prevent excessive movement when the nose is being cleared. Apply the tip of the aspirator (1) to the child's nostril or put it carefully into the child's nose. Put the mouthpiece (3) into your mouth and breathe the air in. When you breathe in the air through the mouthpiece, underpressure is produced, which sucks in the phlegm out of the child's nose. You can adjust the suction power as needed. Having finished sucking away the phlegm, wipe the child's nose with a tissue. Repeat if necessary. If the phlegm is too dense and dry, the aspirator will be less effective. In that case, prior to using the aspirator it is recommended to apply saline or sea salt spray. The aspirator is provided with a feature that prevents the phlegm from getting into the parent's mouth. To feel comfortable, it is also possible to place a bit of cotton wool inside the plastic tube.

Cleaning the aspirator: Disassemble the aspirator as shown in the picture on the packaging. Remove the remains of phlegm from the tip (1) or the plastic tube (2). Clean the disassembled elements with warm water with baby soap or baby bottle cleanser. Rinse thoroughly and dry all the parts.

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