TONIMER

ISOTONIC SPRAY 300 m0sm/kg

EN_INDICATIONS

Sterile isotonic sea water solution with a slightly alkaline pH, indicated to remove and fluidify the mucus in the nasal passages in case of dryness of the mucous membranes and colds, thus facilitating more comfortable breathing. It contains no preservatives and does not disperse propellant (air), as it is not in contact with the product, thus reducing the possibility of allergic reactions.

The product is indicated:

- in TONIMER NORMAL ISOTONIC SPRAY and TONIMER STRONG ISOTONIC SPRAY presentations for adults;
- in TONIMER SOFT ISOTONIC SPRAY presentation for adults with hypersensitive mucous membranes;
- in TONIMER ISOTONIC BABY SPRAY presentation for newborns and children.

The presence of the non-return valve and a sac that isolates the physiological solution:

- guarantees sterility;
- makes it possible to use the bottle in any direction (360°);
- prevents mucus from being sucked into the bottle.

DIRECTIONS FOR USE:


wash your hands before use.

- Once the cap is removed, children who are able to use the product on their own (under adult supervision) may do so by tilting the bottle at any angle. It is best applied by leaning over a washbasin, tilting the head to one side and inserting the spray nozzle into the upper nostril. Press on the nozzle till the product is sprayed for the recommended time (see dispensing times and frequency of use). Repeat the same procedure in the other nostril (drawing 1).
- For newborns and children requiring adult assistance:
 - lie the child on its side, insert the nozzle into the upper nostril and press for the recommended time. Turn the child onto its other side and repeat the same procedure in the other nostril (drawing 2).
- After use, always wash and dry the nozzle and close the container using the cap supplied.

DISPENSING TIMES AND FREQUENCY OF USE:

- TONIMER NORMAL ISOTONIC SPRAY (adults): press on the dispenser for 1-2 seconds/nostril. The recommended frequency is 2-3 applications a day, 3-4 hours apart for 15 days, or according to medical advice.
- TONIMER SOFT ISOTONIC SPRAY (adults): press on the dispenser for 5 seconds/nostril. The recommended frequency is 2-3 applications a day, 3-4 hours apart for 15 days, or according to medical advice.
- TONIMER STRONG ISOTONIC SPRAY (adults): press on the dispenser for 1 second/nostril. The recommended frequency is 2-3 applications a day, 3-4 hours apart for 15 days, or according to medical advice.
- TONIMER ISOTONIC BABY SPRAY (newborns and children): press on the dispenser for 5 seconds/nostril. The recommended frequency is 2-3 applications a day, 3-4 hours apart for 15 days, or according to medical advice.

WARNINGS: for nasal use. For personal use only. Do not use the product if the bottle appears to be damaged. Do not use the product after the expiry date. The expiry date refers to an intact, correctly stored product. Do not exceed the recommended dosage. Do not use the product if you know you are sensitive to any of its components. In rare cases, temporary burning of

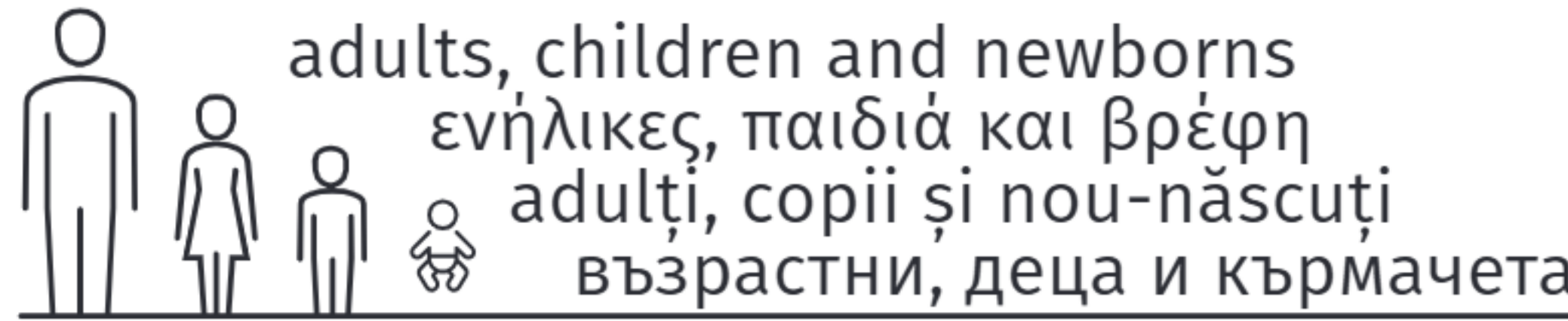the mucous membranes or slight nasal bleeding can occur. This will not compromise the safety and efficacy of the product. TONIMER STRONG ISOTONIC SPRAY (strong jet) should not be used on newborns and children unless advised by a doctor. It is not recommended for adults with particularly sensitive nasal mucous membranes or susceptibility to epistaxis. Stop using the product, consult your doctor and inform the manufacturer if side effects occur. If symptoms persist after treatment, consult your doctor. There are no contraindications related to using the product during pregnancy and breastfeeding. The product should only be used on newborns under paediatric advice. Drugs and medical devices for inhalation must only be taken simultaneously with the product under the supervision of a doctor who can assess possible interactions. In the event of serious accidents related to administration of one of the products from TONIMER ISOTONIC SPRAY line, please send a report to the manufacturer (customercare@ganassini.it), the Italian Ministry of Health or the competent authority of the EU member state or of the country in which the product was used. Avoid contact with the eyes; in the event of contact, rinse with water. Keep the product out of the reach and sight of children. Dispose of the device responsibly after use. **Store with the cap closed, away from heat, in a cool and dry place. Warning** Pressurised container: may burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F. Keep out of reach of children. Do not make use other than those specified.

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ISTITUTO GANASSINI S.p.A. di Ricerche Biochimiche via P. Gaggia, 16 - 20139 Milano - Italy www.ganassinicorporate.com