# Surecheck

EN

## User Information Leaflet

PLEASE READ THE INSTRUCTIONS CAREFULLY BEFORE USE!


*Surecheck pregnancy test can be used as early as the FIRST DAY of a missed period at ANY TIME of the day.*

### Before you begin:


1. Read the instructions carefully.
2. Have a watch, clock or timer handy.
3. Check the expiration date on the box. Do not use if expired.

### How to use:

1. Open the protective foil pouch immediately before use. Remove the test and pipette from the pouch.
2. Urinate in a clean cup, draw 3-4 drops into the pipette and dispense it into the sample well of the cassette.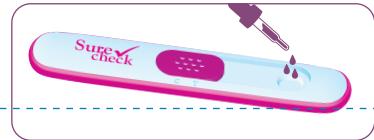

3. Read the result in 3 minutes.

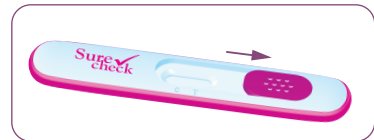
### Results:

NOT PREGNANT

Only one colour line appears in the test window.

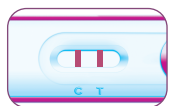PREGNANT

Two colour lines appear in the test window.

**Invalid result:** If no line appears in the control region (C) after 5 minutes, then the result is invalid. Repeat the test using a new unopened Surecheck test kit. The test is rendered invalid if it is removed from the foil pouch for more than 15 minutes before use.

See your healthcare professional if you have a positive result.

### FAQs:

**1. I tested 'Pregnant' but upon testing again I got a 'Not Pregnant' result or my period has started. What does this mean?**

Surecheck pregnancy tests are over 99% accurate from the first day of a missed period. If you get a positive

result, even if it is a pale second line, you can be sure that you are pregnant. However, it is possible to get a positive result and find out later that you are no longer pregnant (you may get a negative result upon checking again or your period might start). Natural loss during the first few weeks of pregnancy, known as 'early pregnancy loss', is unfortunately not uncommon.

### 2. When can I use the test?

Surecheck pregnancy test can be used as early as the first day of a missed period at any time of the day.

### 3. I have an irregular period. How do I know when to do the test?

When your period is irregular you can use Surecheck pregnancy test 19 days after the day you think you might have conceived. If you get a negative result, you should test again in 48 hours.

### 4. What should I do if the line in the test region (T) is very pale? Pale second line means you are pregnant.

### 5. What circumstances may affect the result?

- Ingestion of certain medications may affect the result.
- If you use Surecheck test within 8 weeks of a childbirth or abortion, you may get a false positive result because of the hCG hormone concentration still present in your body. If in doubt, please contact a doctor to interpret the result.
- Drinking large amounts of liquids may lower the concentration of the hCG hormone in the urine and may affect the result.

Birth control pills, antibiotics, analgesics (painkillers) and alcohol will not affect the result.

### 6. Can test results be interpreted after 10 minutes? No. Any results after the 10th minute should be disregarded.

### 7. A pink background color and vertical streaking appeared in the result area during the testing period. Is this a concern?

No. Variations in the chemical composition of the urine sample and the humidity of the room can cause vertical streaking/pink-rose background. It will not affect the test results.

For information about how to use the test, how to read the results or any other questions:

www.surecheck.info, [f](#) /surecheck, [@](#) /surecheck

### BIBLIOGRAPHY

1. Canfield RE, Morgan FJ, Kammerman S, Bell JJ, Agosto GM. Studies of human chorionic gonadotropin. Recent Prog Horm Res. 1971; 27:121-164.
2. Ross GT. Clinical relevance of research on the structure of human chorionic gonadotropin. American Journal of Obstetrics and Gynecology 1977; 129:795

**Specificity:** The specificity of the One Step HCG Urine Test was determined from cross reactivity studies with known amounts of Luteinizing Hormone (hLH), Follicle Stimulating Hormone (hFSH), and Thyroid Stimulating Hormone (hTSH). Negative results were obtained from all tests conducted with 300 mIU/ml hLH, 300 mIU/ml hFSH and 1000 µIU/ml hTSH.

Each Surecheck Cassette box contains: a foil pouch with one Surecheck Cassette pregnancy test, pipette and desiccant (moisture absorbing pouch - do not open!); user information leaflet. [Blue Cross Bio-Medical Co. Ltd.](#), Beijing Industry Zone, PRC 102209. [Shanghai International Holding Corp. GmbH](#) (Europe), Hamburg, Germany. [Keep out of reach of children. Not for internal use. The test is for self-testing. The test is for single-use. Simple to use, no special training required. Keep away from direct sunlight, heat and moisture. Test sensitivity: 25 mIU/ml. Store at a temperature between 4° and 30°C. Importer: Unicomis Switzerland GmbH - Sofia branch, 118 Bulgaria Blvd., Abacus Business Centre, fl. 5, 1618 Sofia, Bulgaria.](#)

Latest revision: 29.09.2022

Surecheck is made for **Unicomis Switzerland GmbH**.

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A Surecheck tesztet **Unicoms Switzerland GmbH** gyártja.