Surecheck

STREAMER

Pregnancy test with over 99% accuracy
Results as early as the first day of a missed period!

EN

User information leaflet

Please read the instructions carefully before use!

The prospect of being pregnant can bring joy or anxiety. In either case, the best thing to do if in doubt is to find out for sure. Surecheck Streamer is developed with the special Colloidal Gold technology to ensure test result ACCURACY above 99% from the FIRST DAY of a missed period – at ANY TIME of the day. The specially engineered sleek design makes Surecheck Streamer more hygienic and easy to use.

ACCURATE AT ALL TIMES

Store at a temperature between 4°C and 30°C.

Simple to use, no special or previous training required for taking the test.

Before you begin 1. Read the instructions carefully. 2. Have a watch, clock or timer handy. 3. Check the expiration date on the box. Do not use if expired. 4. Do not use the test in case of foil pouch damage.

HOW TO USE

INTERPRETATION OF RESULTS

NOT PREGNANT

Only one colour line appears in the test window.

PREGNANT

Two colour lines appear in the test window.

Invalid result

- If no line appears in the test window after 5 minutes, the result is invalid. Repeat the test using a new Surecheck Streamer test kit.
- The test is rendered invalid if it is removed from the foil pouch 15 minutes or more before use.

Surecheck Streamer should not be reused!

See your healthcare professional if you have a positive result!

FAQs:

1. I tested 'Pregnant' but upon testing again I got a 'Not Pregnant' result, or my period has started. What does this mean?
Surecheck pregnancy tests are over 99% accurate from the first day of a missed period. If you get a positive result, even if it is a pale second line, you can be sure that you are pregnant. However, it is possible to get a positive result and find out later that you are no longer pregnant (you may get a negative result upon checking again, or your period might start). Natural loss during the first few weeks of pregnancy, known as 'early pregnancy loss', is unfortunately not uncommon.
2. When can I use the test? Surecheck pregnancy test can be used as early as the first day of a missed period, at any time of the day.
3. I have an irregular period. How do I know when to do the test?
When your period is irregular you can use Surecheck pregnancy test 19 days after you think you might have conceived. If you get a negative result, you may test again in 48 hours.
4. What should I do if the line in the test region (T) is very pale?
Pale second line means you are pregnant.
5. What circumstances may affect the result?
 - Ingestion of certain medications may affect the result.
 - If you use Surecheck test within 8 weeks of a childbirth or abortion, you may get a false positive result because of the hCG hormone concentration still present in your body. If in doubt, please contact a doctor to interpret the result.

- Drinking large amounts of liquids may lower the concentration of the hCG hormone in the urine and may affect the result.
- Birth control pills, antibiotics, analgesics (painkillers) and alcohol will not affect the result.
- 6. Can test results be interpreted after 10 minutes?
No. Any results after the 10th minute should be disregarded.
- 7. A pink background color and vertical streaking appeared in the result area during the testing period. Is this a concern? No. Variations in the chemical composition of the urine sample and the humidity of the room can cause vertical streaking/pink-rose background. It will not affect the test results.

For information about how to use the test, how to read the results or any other questions: www.surecheck.info, [f /surecheck](https://www.facebook.com/surecheck), [@surecheck](https://www.instagram.com/surecheck).

BIBLIOGRAPHY

1. Canfield RE, Morgan EJ, Kammerman S, Bell JJ, Agosto GM. Studies of human chorionic gonadotropin. Recent Prog Horm Res. 1971; 27:121-164.
2. Ross GT. Clinical relevance of research on the structure of human chorionic gonadotropin. American Journal of Obstetrics and Gynecology 1977; 129:795.

Specificity: The specificity of the One Step HCG Urine Test was determined from cross reactivity studies with known amounts of Luteinizing Hormone (hLH), Follicle Stimulating Hormone (hFSH), and Thyroid Stimulating Hormone (hTSH). Negative results were obtained from all tests conducted with 300 mIU/ml hLH, 300 mIU/ml hFSH and 1000 µIU/ml hTSH.

Each Surecheck Streamer box contains: foil pouch with one Surecheck Streamer pregnancy test; desiccant (moisture absorbing pouch; do not open!); user information leaflet. Blue Cross Bio-Medical Co. Ltd., Beiqijia Industry Zone, PRC 102209. www.surecheck.info Shanghai International Holding Corp. GmbH (Europe), Hamburg, Germany. Keep out of reach of children. Not for internal use. The test is for self-testing. The test is for single-use. Simple to use, no special training required. Keep away from direct sunlight, heat and moisture. Test sensitivity: 25 mIU/ml. Store at a temperature between 4° and 30°C. **Importer:** Unicoms Switzerland GmbH - Sofia branch, 118 Bulgaria Blvd., Abacus Business Centre, fl. 5, 1618 Sofia, Bulgaria.

Surecheck is produced for
Unicoms Switzerland GmbH

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